



Intelligent Device System

The intuitive design and user-friendly set-up of the HA300 Electrosurgical Generator (diathermy) for use in the modern operating room, it features both monopolar and bipolar functions to satisfy all the surgeon's demands with safety, flexibility, reliability and convenience.

REM (Return Electrode Monitoring)

The REM system continually monitors patient impedance levels and deactivates the generator if a fault in the patient/return electrode contact is detected, which minimizes the danger of burning incidents.




Memory Records Features

Memory program that allows customize settings for different interventions and surgeons. Now total from 0~99 memory locations can be available for user-defined settings, which make Orion a versatile, indispensable companion in HF surgery.

Real-time Monitoring and Instance Response System for Tissue Density

This proprietary technology delivers optimal clinical effects through continuous synchronization of current and voltage. It samples the current and voltage 450,000 times per second which enables it to respond to tissue impedance changes in less than 10 milliseconds, which ensuring the machine achieves optimum energy output levels quicker and more accurately – ensuring that only the precise voltage needed is safely delivered to each tissue type.

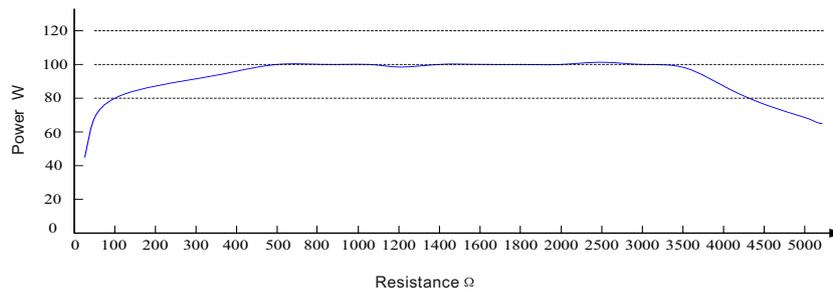

Two pencils Work Simultaneously (Optional)

It can meet the special Surgeries like Heart Bypass operation which ensures two users can operate respectively without interference

Automatic Start/Stop with Alarm (Optional)

Under Bipolar Cut & Coag modes, user can choose Pedal Control Or AutoStart Control

Compatible with Other Equipments:

Argon Plasma Coagulation System
Optimum Smoke Evacuation System

Wireless Footswitch Facility (Optional)

Wireless 3 Pedal Footswitch For Monopolar & Bipolar Modes

Application of Use

General Surgery; Gastroenterology; Dermatology;
Vascular Surgery; Obstetrics and Gynecology Heart/Thoracic Surgery; ORL/ENT;
Minimally Invasive Surgery (MIS) Cerebral surgery Neurosurgery;
Orthopedics & Plastic Surgery, Trans Urethral Resection (TuR) and etc.

Technical Specification:

Operating Frequency : 480kHz
Power Supply: 100-240V / 50-60Hz

Weight and Dimension:

Weight: 4.00KG
Dimension: 380 x 305 x 115 mm

High Frequency Leakage:

Monopolar: < 150mA; Bipolar <60mA

Low Frequency Leakage:

Normal Polarity, Intact Chassis Ground: <10μA
Normal Polarity, Ground open < 50μA
Reverse Polarity, Ground Open < 50μA
Sink Current, 140V Applied, All inputs < 50μA

Technical Specification

Working Mode	Maximum P-P Voltage	Rated Load	Max. output power	Crest Factor (Typical)	
Monopolar Cut	Pure	1700	300Ω	400W	1.5
	Blend 1	2100	300Ω	250W	2
	Blend 2	2500	300Ω	200W	2.6
	EndoCut	1700	300Ω	99W	1.5
Monopolar Coag	Spray	6000	500Ω	120W	8.0
	Fulgurate	5000	500Ω	150W	7.0
	Dessicate	4000	500Ω	150W	5.0
Bipolar	Micro	360	100Ω	80W	1.5
	Macro	670	100Ω	80W	1.5
	CUT	700	100Ω	99W	1.5